



Strong Biotech Corporation



Strong Helicotec UT[®] Plus

Catalog No.HUP01 **50** tests

Paper Rapid Urease Test detects *H. pylori* in gastric biopsy

■ **How it works:**

H. pylori produce large amounts of the enzyme urease. The *Helicotec UT Plus* test paper detects pH shift and changes colour resulting from the breakdown of urea by urease into ammonia.

■ **Easy to use:**



Peel the label



Transfer the biopsy onto the test paper



Re-seal and squeeze



Record, the date, time and patients info.

■ **Clear results within 5 minutes to 1 hour**

Reaction Time	5 mins	30 mins	1 hour	1 hour	1 hour	1 hour
Color of Ring	Magenta	Magenta	Magenta	Dark Pink	Light Pink	Yellow
Urease Activity	++++	+++	+++	++	+	-

■ **Benefits:**

- Easy-to-Use Easy-to-Store
 - High Sensitivity and Specificity
- Clear, Rapid and Accurate Results
 - Superior Quality
- Competitive Price

■ **Storage:**

Helicotec UT Plus is stable for 24 months at room temperature in a cool and dry place.

■ **Notes:**

In most cases, positive results will appear in 5 to 30minutes, while weakly positive results may take as long as an hour to develop. This test is for in vitro diagnostics only, and should be performed by a physician or medical technician.



C.R. Kennedy(NZ) Ltd
3 Hotunui Drive,
Mt Wellington, Auckland.
Tel. 09 276 3271
Fax. 09 276 9645
medical@crknz.co.nz